


Double Coated

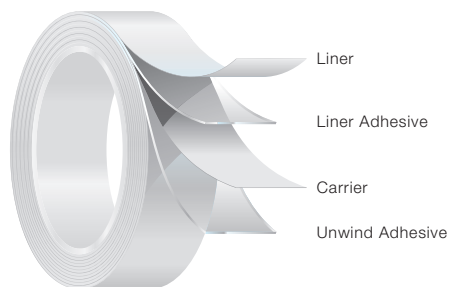

# Double Coated Polyester Tape

## Typical Applications

Designed with self-adhesive edges for surgical drapes while providing excellent adherence to both skin and hydrophobic fabrics and for device mounting to skin.

## Surgical

Spec # 58247

### Product Profile

(not for specification use) Test Methods: PSTC-33

		US Mils	MM's
<b>Unwind Adhesive</b>	Solventless medical acrylic	2.4	0.06
<b>Carrier</b>	Polyester		
<b>Liner Adhesive</b>	Solventless medical acrylic	2.4	0.06
<b>Liner</b>	White super calendared glassine	2.6	0.07
	Total Construction	7.4	0.19

## Features & Benefits

- Breathable
- Transparent
- Moisture Resistant
- Elastic
- Conformable
- Die-cuttable
- High Initial Tack
- Gamma Sterilization
- EtO Sterilization

### Physical Properties

(not for specification use)

#### Adhesive Properties

##### Peel Adhesion

Test Method(s): PSTC-3, ASTM D-3330, STD-10

2 mil PET 90° 12" / min

Substrate	Test Side	Lbf / in	US Oz / in	N / Meter
PE	Liner	0.9	14	150
	Unwind	0.9	14	150

##### TLMI Release

Test Method(s): PSTC-4, STD-8

Product 90° 300" / min

Test Side	Gf / 2 in w	G / Meter
Liner	73	1440

##### Tack

Test Method(s): PSTC-5, STD-7

2 mil PET

Substrate	Test Side	Lbf / in	US Oz / in	N / Meter
SS	Liner	5.3	86	936
	Unwind	6.5	105	1144

- High
- Medium
- Low

## MED 6361U

Electromedical | Ostomy | **Surgical** | Wound Care**Applications Information**

- Suitable for direct contact to skin application.
- Specially designed, ethyleneoxide and gamma-ray sterilizable pressure-sensitive adhesive.
- Good adhesion to all common natural and synthetic fabrics.
- Adhesives are pure solvent-free acrylic polymers.
- Polyester carrier.
- Clear adhesive has no discernible odor.
- Good initial bond.

**Storage and Shelf Life**

Two years when stored at 70°F (21°C), 50% relative humidity, out of direct sunlight, in original packaging.

**Safety**

The adhesive used on MED 6361U is rubber based. This adhesive has been tested by an independent laboratory on humans as well as animals. The results indicate that the adhesive is non-irritating and non-sensitizing in humans. The tests run on animals were in accordance with Title 16 of the code of Federal Regulations, paragraph 1500.41. The Repeat Insult Patch Test was used to determine sensitization in humans and to indicate potential safe use of MED 6361U in medical devices. Good Manufacturing Practice places the burden of safe product performance on the manufacturer or processor of the device. If further information is required regarding the safety of Avery Dennison Specialty Tape Division medical products, contact your sales representative or our Medical Products Department.

**Limited Warranty**


All statements, technical information and recommendations concerning products sold or samples provided by AVERY DENNISON are based upon tests believed to be reliable but do not constitute a guarantee or warranty. All products are sold and samples of products provided with the understanding that PURCHASER has independently determined the suitability of such products for its purposes. AVERY DENNISON warrants the products to be free from defects in material and workmanship. Should any failure to conform to this warranty appear within one year\* after the initial date of shipment, AVERY DENNISON shall, upon notification thereof and substantiation that the products have been stored and applied in accordance with AVERY DENNISON's standards, correct such defects by suitable repair or replacement without charge at AVERY DENNISON's plant or at the location of the products (at AVERY DENNISON's election) provided, however, if AVERY DENNISON determines that the repair or replacement is not commercially practical, AVERY DENNISON shall issue credit in favor of PURCHASER in an amount not to exceed the purchase price of the products.

THIS WARRANTY IS EXCLUSIVE AND IS IN LIEU OF ANY IMPLIED WARRANTY OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR OTHER WARRANTY OF QUALITY, WHETHER EXPRESS OR IMPLIED, EXCEPT THE WARRANTY OF TITLE AND AGAINST PATENT INFRINGEMENT. NO WAIVER, ALTERATION, ADDITIONS OR MODIFICATIONS OF THE FOREGOING CONDITIONS SHALL BE VALID UNLESS MADE IN WRITING AND MANUALLY SIGNED BY AN OFFICER OF AVERY DENNISON.

\*Or in the time period stated on the specific product specification sheet, if any, and if not then on the specific product information literature in effect at time of shipment.

**Limitation of Liability**

IN NO EVENT SHALL AVERY DENNISON BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES, INCLUDING BUT NOT LIMITED TO, LOSS OF PROFIT, LOSS OF USE OF PRODUCTION OR LOSS OF CAPITAL. THE REMEDIES OF PURCHASER SET FORTH HEREIN ARE EXCLUSIVE AND THE TOTAL LIABILITY OF AVERY DENNISON WITH RESPECT TO ANY CONTRACT, OR ANYTHING DONE IN CONNECTION THEREWITH SUCH AS THE PERFORMANCE OR BREACH HEREOF, OR FROM THE MANUFACTURE, SALE, DELIVERY, RESALE, INSTALLATION OR USE OF PRODUCTS WHETHER ARISING OUT OF CONTRACT, NEGLIGENCE, STRICT LIABILITY, TORT, OR ANY UNDER LEGAL THEORY OR UNDER ANY WARRANTY, OR OTHERWISE, SHALL NOT EXCEED THE PURCHASE PRICE OF THE PRODUCTS UPON WHICH THE LIABILITY IS BASED.

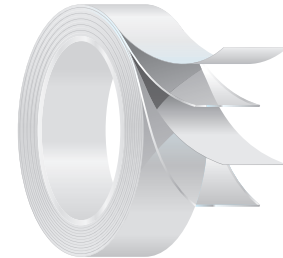

Double Coated  
Polyester Tape

**MED 6361U**

If you require a larger sample size, please contact your Account Manager or Customer Service.